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Class I Type B (IEC 60601-1)

Manufactured By NOVIN Electronics & Electrical Industries Co.
No. 169, Electrical & Electronics Industrial Zone of Khomeinishahr, Esfahan, Iran
www.novinmed.com

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Legal Responsible Name of Company

Name of Company: OBELIS S.A

Name of responsible person: Gideon Elkayam

Address: Boulevard General Wahis 53, 1030 Brussels, BELGIUM

Tel: +32.2.732.59.54

Fax: +32.2.732.60.03

E-Mail: mail@obelis.net

Manual Purpose

This manual provides the instructions necessary to operate the device HPL 885G based on its intended use. It also describes all adjustable parameters by the device, as well as contraindications and possible side effects of medical laser therapy. The observance of this manual is a prerequisite for proper operation and assures patient and operator safety. If you have any questions about the operation of the device HPL 885G, please contact our customer service department.

Intended Audience

This manual is provided for physiotherapists. The physiotherapists are expected to have the essential working knowledge and skill regarding the Laser therapy modality.

TABLE OF ONTENTS

Device General Description.....	7
Used Signs.....	8
Indications.....	9
Contraindications.....	10
Warnings and Safety Information.....	13
Components and Controllers Explanation.....	16
Installation.....	20
Device Operation.....	22
Therapeutic Protocols.....	36
Warranty and Responsibilities.....	37
Safety Notes.....	38
Therapeutic advice.....	39
Repair And Maintenance.....	40
Maintenance.....	40
Laser Protector Window.....	41
Calibration Socket.....	42
Repair and Troubleshooting.....	43
How to open the fuse box.....	44
Recycling the Device.....	44
Device Specifications.....	45
Standards and Classification.....	46
Environmental Conditions.....	47
EMC Declaration.....	48
Symbols and Labels.....	51
Symbols.....	51
Labels.....	54
Device Accessories.....	56

INTRODUCTION

NOVIN Electronics & Electrical Industries feels honored to have been a pioneered physiotherapy Equipment manufacturer and a trustworthy attendant of *community physiotherapists since 1988*. Every year, Novin Company produces various equipment according to the new world technologies in different aspects like electrical stimulator, magnet therapy, laser therapy, etc. this is its professional team activities in researching, designing, creating and manufacturing parts. Now, we achieve our ultimate goals such as improving product quality, reasonable prices and timely delivery of products after years of effort and perseverance.

The device High power Laser 885M is designed and manufactured in Grand series product. The maximum laser power of this device is 15 Watt and the wavelength of output laser is 810 and 980 nm at the same time.

The device provides 57 therapeutic protocols, for each of which supplementary information, as well as pictures of the therapy area, are provided. The use of 10-inch, colored, touch screens in addition to optimized interface designation brings about, even more, convenience and satisfaction of users.

It is hoped that this device could meet the needs of physiotherapists.

Warning: Read this user manual carefully before starting up the device

DEVICE GENERAL DESCRIPTION

The device High Power Laser 885G is a laser therapy device with a maximum output power of 15 watt that simultaneously emits 810 and 980 nm wavelengths at its output; Laser emission is possible in a variety of modes, including single, burst, pulsed, and continuous. For the sake of user convenience, two Spacers of 25 and 35 mm are provided that their connection to the tip of the applicator can increase the area, to which extent emission takes place and subsequently decrease the therapy time.

This device has 57 therapeutic protocols, for each of which, in addition to the value of therapy parameters, supplementary information and placement images of the therapy area are provided. Additionally, there are some particular options such as sources of the most useful anatomy images and a range of device customization like choosing between varieties of graphical themes, selecting the size of the text and so forth.

USED SIGNS

Hazard: The risk of a hazard that may result in death or serious injury if the safety instructions are ignored.

Warning: The risk of a hazard that may result in physical injury if the safety instructions are ignored.

Caution: The risk of a hazard that may result in the device or adjacent devices defects if the safety instructions are ignored.

Note: Information on the safe function of the device and its accessories.

Technical explanation: Technical information about the device.

Hint: Useful hints and guides

Definition: Definition of phrases used in the text.

INDICATIONS

- Tendonitis like rotator cuff, hip, knee, and Achilles tendonitis.
- Bursitis
- Osteoarthritis
- Frozen shoulder
- Myofascial trigger points
- Tennis elbow (lateral epicondylitis)
- Carpal tunnel syndrome
- Fasciitis plantar
- Muscle spasm
- Wound healing
- Shingles
- Cervical and lumbar spondylosis
- Lower extremity strains
- Fibromyalgia
- Dupuytren's contracture
- Piriformis syndrome
- Tibial stress syndrome

CONTRAINDICATIONS

- Heart and its surroundings in patients with heart disease
- Areas sensitive to light
- Infected areas
- Bleeding tissues
- Projection over eyes
- Gland tissues (such as thyroid, testes, ...)
- Children's growth plate
- Acute inflammation
- Tumors and areas suspected of malignancy
- Pregnancy

Precautions

Additional precaution should be used when the laser light is used on patients with the following conditions:

- Over an area of the spinal cord following a laminectomy, i.e., when major covering tissues have been removed,
- Overnumbed part of the body by local anesthetics,
- On patients with hemorrhagic diathesis

Effects

Physiological Effects

- Expedite healing process and cellular growth
- Expedite and facilitate the process of healing wounds
- Prevent from forming fibrosis
- Cut back on inflammation
- Relieve pain
- Improve cardiovascular performance
- Improve metabolic function
- Improve neural function

Side Effects

Although therapeutic High Intensity Laser is generally safe to use and promotes reparative processes, it can still damage the eye and so protective eyewear is obligatory throughout the therapy. With the increasing power of High Intensity Laser, caution is in place as to its thermic effects which on one hand provide very pleasant therapy, but on the other, if not monitored and used by a trained operator, it can damage the tissue.

BEFORE START

Therapists should follow the steps below to apply a treatment:

A) Preparation of the Generator

1. If applicable, clean the laser lens with an approved cleaner and/or polish.
2. Choose a predefined protocol or adjust the treatment parameters manually according to disease. The adjustable parameters are peak power, average power, mode, dose, frequency, and area.
3. If the therapeutic protocols are used, see the complementary information and the placement images which are useful for finding the appropriate area for therapy.
4. Examine the skin for any wounds and clean the skin.
5. If laser is being applied to open wounds, cover the wound surface with a clear plastic wrap to prevent transmission of contaminants. This technique results in a loss of approximately 8% of the total energy.
6. Select the desired output power on display.

B) Preparation of the Patient

1. Assure that the patient is free on any contraindications to the application of High Level Laser Therapy (HLLT).
2. The patient and clinician shall wear goggles.
3. Clean the area to be treated with soap and water or alcohol swabs. Allow the area to dry thoroughly before initiating the treatment.
4. Some patients are more sensitive to laser output and may experience a reaction similar to a heat rash. Be sure to inspect the treatment area during and following the treatment, and discontinue if an adverse reaction does occur.
5. Higher output levels have a greater potential for patient discomfort. Choose a lower dosage to reduce output or select a pulsed duty cycle to decrease patient discomfort.

WARNINGS AND SAFETY INFORMATION

Warning: Read the user manual carefully before operating the device HPL 885G.

Warning: To avoid the risk of electric shock, the device must only be connected to the supply mains with protective earth.

Warning: The device HPL 885G is intended to be used by a physiotherapist.

Warning: Before treatment, be sure that the patient is not a case of contraindications or precautions.

Warning: The device and accessories shall be disposed in compliance with relevant regulations. Contact your municipality to check where you can safely dispose the device.

Warning: Maintenance and cleaning of the device should be conducted according to the manufacturer's recommendations in the user manual.

Warning: Useful lifetime of the device is 10 years. It is recommended to calibrate the device once a year.

Warning: Do not modify this equipment without authorization of the manufacturer.

Warning: Any treatment instructions regarding treatment location, duration, and intensity require physical therapy knowledge about High Power Laser Therapy (HPLT).

Warning: To avoid the effects of electromagnetic interference, never use the device within 1 meter of short wave or microwave diathermy devices.


Warnings and Safety Information

Warning: When in operation, the device must be positioned in such a way that direct access to the device's mains power cord gets possible so that it can be disconnected from the mains at any time.

Warning: All cables must be protected against jamming or other mechanical damage.

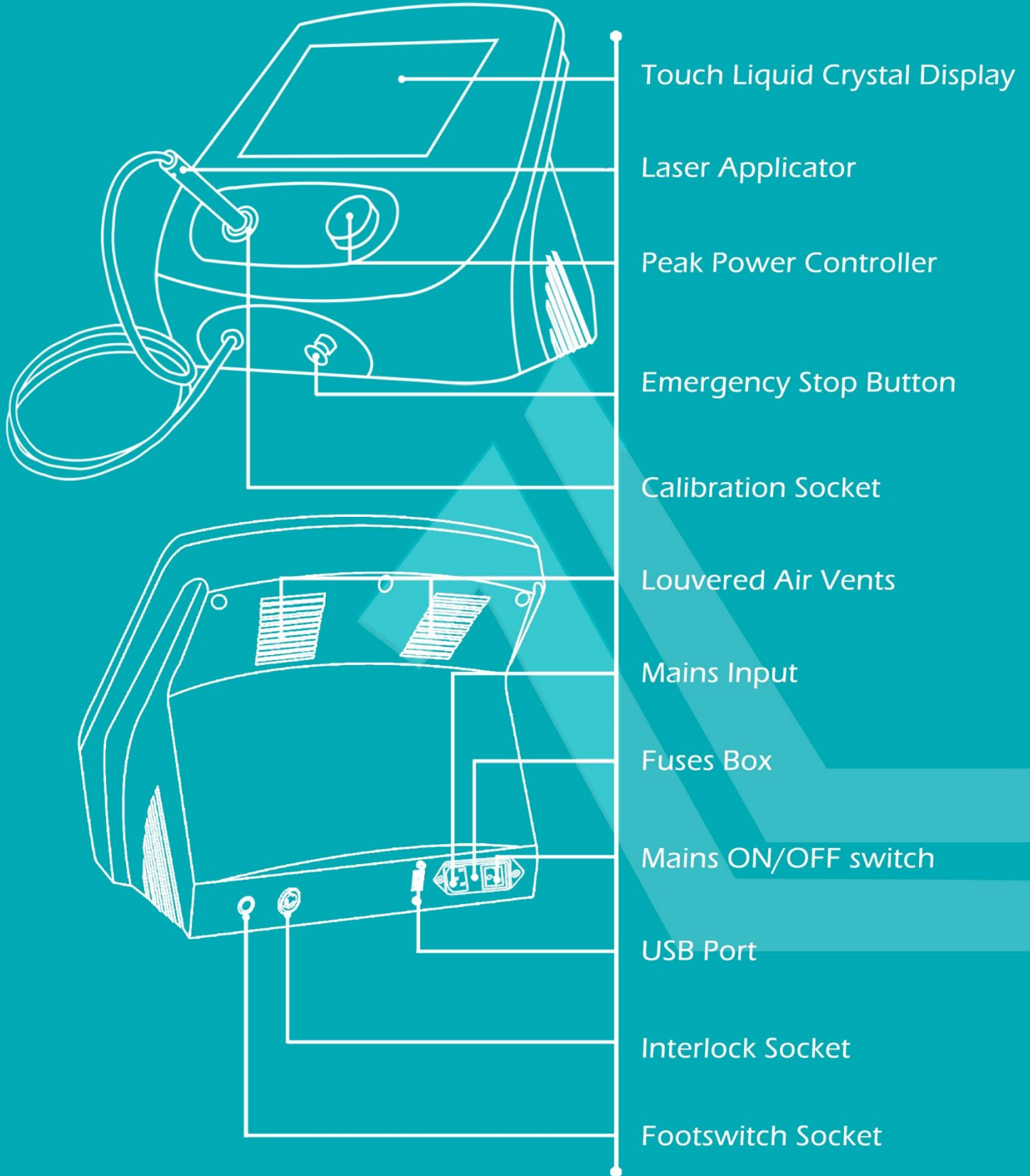COMPONENTS AND CONTROLLERS EXPLANATION

Touch Liquid Crystal Display

All therapy parameters, as well as information regarding how to use this device, will be shown on this screen.

Laser Applicator

Execute the therapy by placing the laser applicator over the target area.

Peak Power Controller

You may adjust the output peak power (per mW) by rotating this controller.

Emergency Stop Button

In case of hazard, use this button to immediately stop the output; Never use this button in normal situations for stopping or pausing the device; This button is designed for emergency conditions only and frequent use can harm the device.

Hint: Before starting up the system, make sure the emergency stop button is not pushed otherwise the device won't start. In this case, release it and push the on/off button thereafter.

Calibration Socket

In order to test the calibration, position the tip of the Handpiece here.

Caution: Do not activate the laser emission by pushing the Start button or pressing the foot pedal when the applicator is inside the calibration socket unless you're executing the calibration test, which lasts a short time of around 10 seconds.

6

Louvered Air Vents

These windows are provided to let out excess heat. Don't block them!

7

Mains Input

The power cable may connect to this part.

8

Fuses Box

Here are the 2.5A fuses of the input power supply.

Mains ON/OFF switch

9

Use this switch to turn the device on or off. Before unplugging the device from the power supply, use this button to turn it off.

USB Port

10

Essential data for updating the software or adding new therapeutic protocols will be saved on flash memory, and this memory will be connected to the device through UBS port, thereafter.

11

Interlock Socket

Interlock can be connected to the device through this socket.


Hint: Before starting the output emission, make sure the Interlock is connected. In fact, Interlock is a hardware lock that prevents incompetent people from using it. If the Interlock is removed from the corresponding socket, turning on the device and activating the output will be impossible.

12

Footswitch Socket :

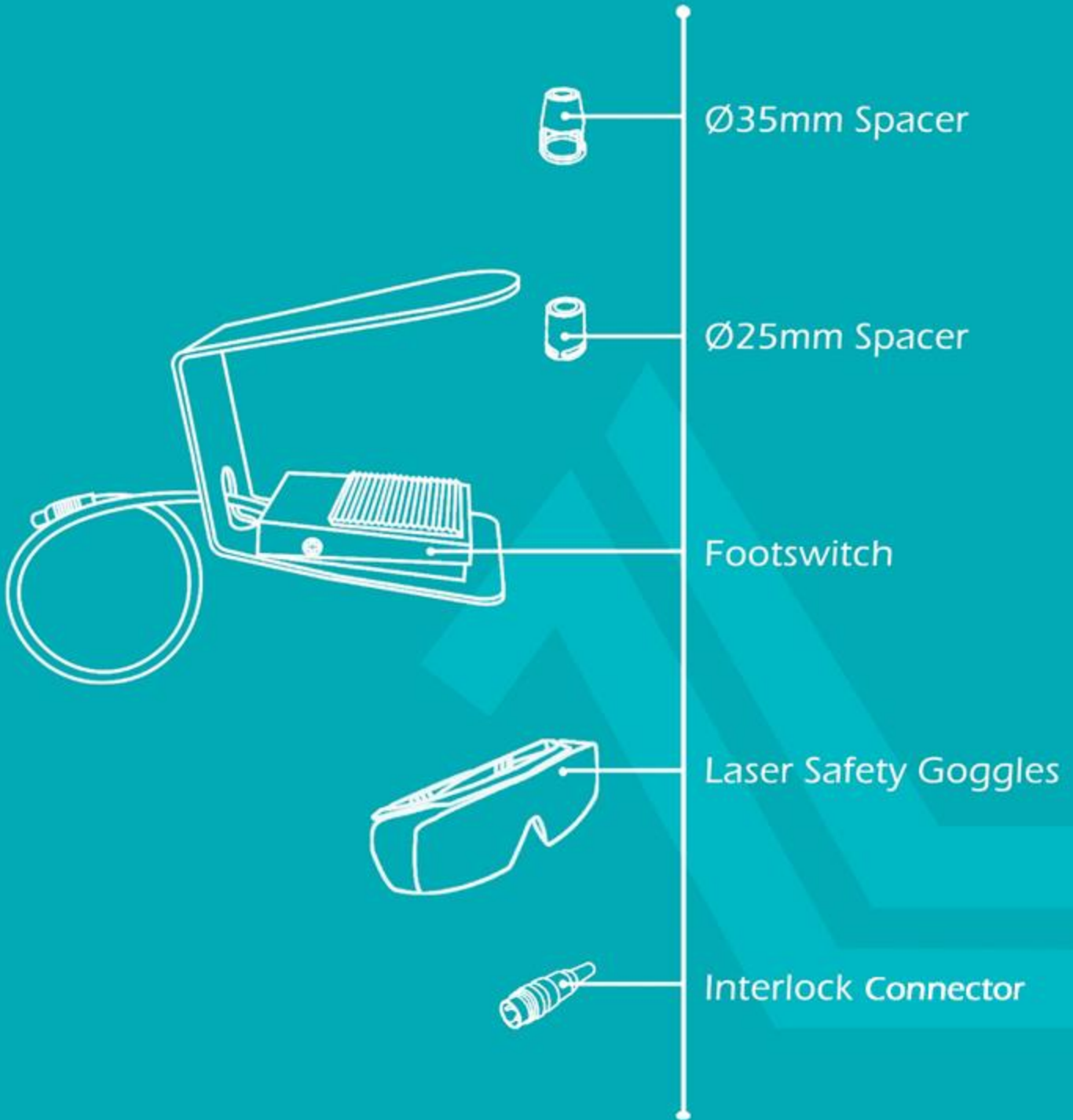Footswitch can be connected to the device through this socket.

COMPONENT, ADJUSTERS AND DISPLAY DESCRIPTION

-  **Ø35mm Spacer**
The spacer of 35mm diameter is used for wider therapeutic area.
-  **Ø25mm Spacer**
The spacer of 25mm diameter is used for more local therapy.
-  **Footswitch**

For applying output, press the footswitch.

-  **Laser Safety Goggles**


The human eye is very sensitive to laser radiation and can be damaged from direct or reflected beams. Laser safety goggles are designed to reduce the amount of incident light of specific wavelength to safe levels, while transmitting sufficient light for good vision.

-  **Interlock Connector**

It is a hardware lock which is designed to prevent unallowed people from using the device.

INSTALLATION

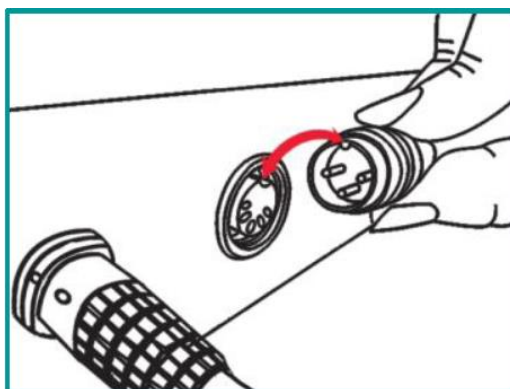1. Unwrap the device package and check its parts for any apparent damage that may have occurred during the post process. If there is any damage, refer it to the company's representative.
2. Place the device in a flat, dry site. Make sure that there is no obstruction in front of the ventilation windows.
3. Connect the Footswitch to its socket on the foreside of the device.
4. Connect the Interlock to its socket in the backside of the device.

5. Make sure that Emergency Stop Button is not pushed. Next, connect the power cable to the device input; Make sure cable is inside the power input all the way to the trapezoidal part. After that, put the ON/OFF button on the ON mode.
6. After turning the device on, wait a while so that the program loads completely and the password request page turns up. Insert the default password, i.e. "0000", and push OK so that the Home Page appears.
7. In the Home Page, choose Clinical Protocols option at first, and thereafter choose a body part, and after that the name of your desired protocol, from the list.
8. If you intend to exert a therapy using arbitrary parameters, choose Manual Operation in the Home Page.
9. Next, adjust the value of any parameter as you wish, in the Therapy page (Parameters such as mode, peak power, average power, dose, frequency, and area).
10. Connect one of the spacers to the applicator tip. Take note that the connectivity of the Spacer to the tip of the applicator occurs via magnet, hence to detach the applicator an amount of force is needed.
11. On the Therapy page push the start button  so that the output goes into ready mode. In this state, a flashy yellow sign , indicating the ready-to-operate mode, will appear on the screen. The guiding light at the applicator output also starts illuminating. At this stage, the laser may be activated by pushing the

Footswitch; at this stage, the warning alarm will play and the flashy red sign  will appear on the screen. By releasing the Footswitch, laser emission will stop and output goes into ready-to-operate mode again.

12. Once finished exerting therapy turn off the device using the ON/OFF button.

13. Unplug the power supply.

Warning: make sure the patient and the medical therapist both use Laser Safety Goggles.

Warning: operating this device within less than 1 meter distance from short wave device or radiofrequency devices that cause electromagnetic interference, is prohibited.

Warning: Use outlets with earth grounding, to deal with the hazard of electric shock. If you doubt whether there exists earth grounding or not, do not plug the power.


Caution: It is advised that only use Novin accessories when operating this device.

DEVICE OPERATION

LOADING

As soon as the device starts, loading pages will appear. Once the appearance of these pages ended, wait for the program to fully load and the password request page to turn up. If the password is inserted correctly, the default password is "0000," the Home page will show up.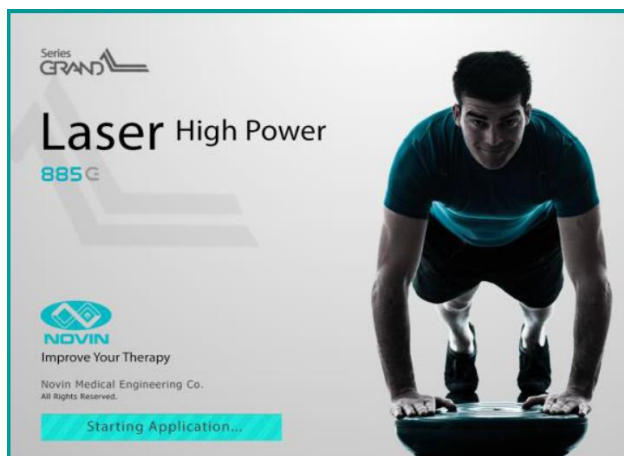

HOME

In the middle of the Home page, there are four main options to choose:

Manual Operation: for therapeutic protocols with arbitrary parameters.


Clinical protocols: a list, including all therapeutic protocols, sorted in alphabetic order.

Body Parts: shows therapeutic protocols separately based on the therapy area.

User Programs: lists programs previously saved by the therapist.

On the right side of the Home window, there is a control bar, comprised of 5 buttons: Help (?), Settings (⚙️), Home (🏠) (return to Home), Back (⬅️) (return to the previous window) and Stop (⏹️) (stop the output and the therapy).

On the top left of the home page, the title of the window (Home) is shown. You may change the title to an option of your choice, in the "Settings/Display/Therapist Name" section.

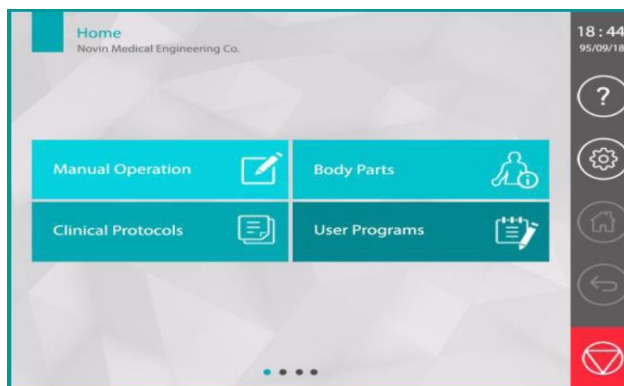
Device Operation


In the Home window, if you swipe your hand to the right or left, other windows with different titles will appear, namely, “favorites” and “Recently Used.” Inside the window “Recently Used,” there exist six last programs that the user has exerted.

Likewise, inside the two Favorite windows, the user’s favorite programs are listed. User may define up to 12 useful programs, with the aim of his convenience.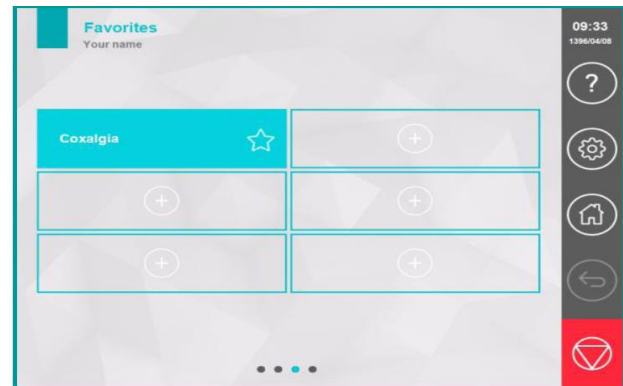


Select one of the 12 buttons in the Favorites windows, to define a favorite program. A window with the title “Add to favorites” will appear, consequently. Choose your favorite program between therapeutic protocols or those saved by the user.

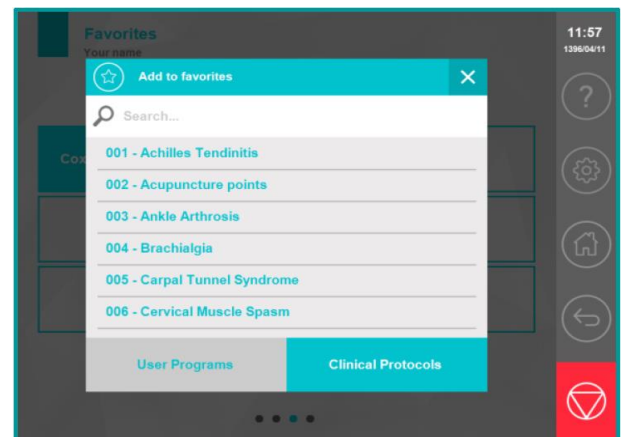
Hint: to erase a program from the favorite list, hold your finger on it for 3 seconds. In the opening window tap on YES to clear the program.

Manual Operation

Selecting this item, in the Home Page, opens the Therapy window in User Defined mode. You may adjust therapy parameters as you wish and exert the program.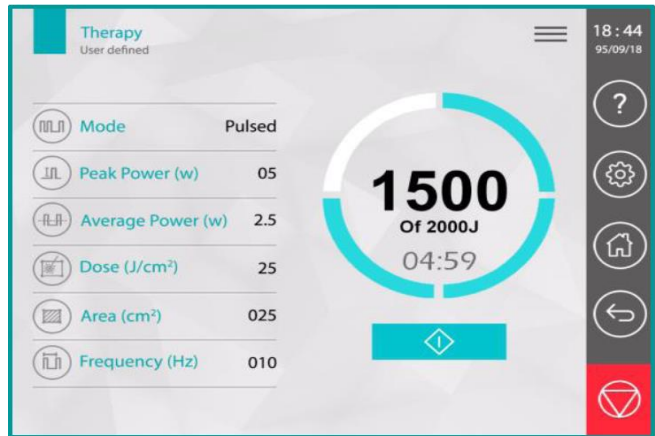

Touching any of the parameters opens a window for value adjustment of that parameter. You may choose a value for each parameter either by swiping your finger upward or downward or use the up or down arrow buttons to find your desired value.

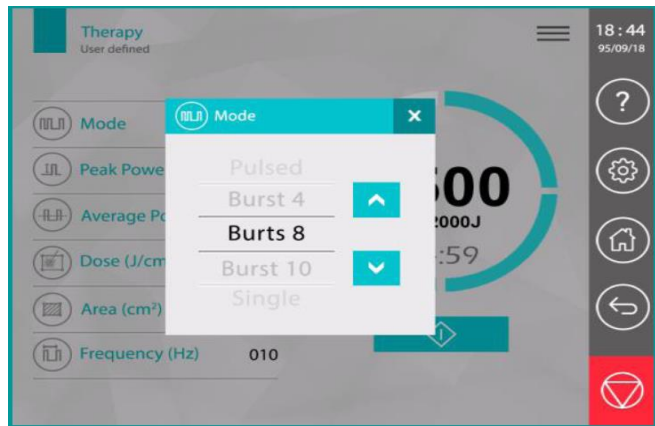Pay attention that to adjust the output peak power, besides touching the Peak Power (mW) button  in therapy panel, you may also use the rotary button on the body of device.

PARAMETER ADJUSTMENT

Mode: modes of laser emission could be selected via this button. Possible options are as follows:

Continuous, Pulsed, Burst4, Burst8, Burst10, and Single.

Continuous: constant laser emission. If this mode is selected, parameters adjustment for Frequency and Average Power will be inactive.

Pulsed: Pulsed laser emission.

Burst10: In this mode, the therapist has to touch the laser Footswitch button, after pushing start in the Therapy page, and hold it up for 10 seconds; this makes the output active which emits ten pulses of 1 Hz. When the device is done emitting 10 pulses, it will go in ready-to-operate mode again; at this stage, user has to release the laser emission footswitch and touch it again so that 10 further pulses are emitted.

Hint: after the device is done emitting 10 pulses, keeping the Footswitch pushed won't emit any further pulses; it is required that after every 10 seconds the Footswitch is released and pushed again.

Burst8: In this mode, every push on the Footswitch, for 8 seconds, emits 8 pulses at 1 Hz frequency.

Burst4: In this mode, every push on the Footswitch, for 8 seconds, emits 8 pulses at 1 Hz frequency.

Single: In this mode, every push on the Footswitch, for 1 second, emits 1 pulse at 1 Hz frequency.

Note: in Burst or Single mode, the button for the Frequency parameter will be inactive.

Peak Power (mW): Using this button the output peak power (per Watt) can be adjusted.

Average Power: Using this button the average power (per Watt) can be adjusted.


Note: in continuous mode, the button for Average Power is inactive since the peak and average values of output power are equal in this mode.

Dose (J/cm²): Using this button, you may adjust the energy density (per J/cm²).

Frequency (HZ): Using this button you may adjust the pulse repetition frequency (per Hz).

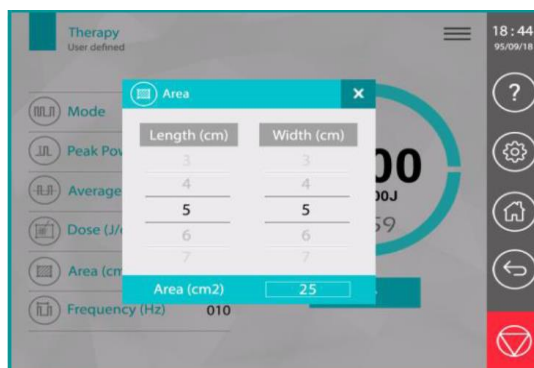Area: selecting this button, opens a window for adjustment of therapy area. Inside this window, the user is required to select the length and width of the therapy area; therefore, assuming the therapy area as a square or rectangle, the area will be calculated as the byproduct of length and width.

Exerting the Therapy

In the Therapy page push the Start button  so that the output goes into ready-to-operate mode; At this time, a flashy yellow sign , indicating the ready-to-operate mode, appears on the screen, after which laser emission may be activated by pushing the Footswitch; this is when warning alarm is played and the flashy red alert sign  appears on the screen. By releasing the Footswitch, laser emission is stopped and output goes into ready-to-operate mode again.

During laser emission, inside the circle on the right of the Therapy window, time is displayed in decreasing order by steps of one second. The amount of energy applied also increases proportionally to the average power and the elapsed time.

Equations used for calculation of the time of therapy, total energy and applied energy:


$$\text{Total Energy (J)} = \text{Area (cm}^2\text{)} \times \text{Dose (J/cm}^2\text{)}$$

$$\text{Treatment Time (sec)} = \frac{\text{Total Energy (J)}}{\text{Average Power (W)}}$$
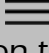

$$\text{Applied Energy (J)} = \text{Elapsed Time (sec)} \times \text{Average Power (W)}$$

The Stop button  on the control bar at the right may be used to stop the output. Doing so stops the laser emission and therapy procedure.

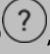Hint: In the Hide menu  if Lock is selected, all parameter adjustment buttons, as well as the buttons on the control bar, will be locked, except the stop button  that will still remain useable for stopping the output when needed. Keep your finger on the Lock button for few seconds to unlock again.

Hint: In ready-to-operate mode and when the output is active, it's impossible to exit the Therapy window using Home  and Back  buttons. In addition, the possibility of selecting Help , Settings , Save , Placement  and Information  become nonexistent and they will turn into inactive.

Hint: It is impossible to alter the Mode, Frequency and Area parameters in ready-to-operate mode, and all these parameters will emerge as inactive.

Hint: Whilst casting laser, it's impossible to alter any of the therapy parameters and hence their respective buttons will emerge as inactive. If you tended to reduce the amount of power, it would suffice to release the Footswitch; therefore, the device will go on ready-to-operate mode, where you can change the amount of power as you wish, and then push the Footswitch again to emit the laser with adjusted power.


Body Parts

If the Body Parts option is selected in the Home window, a window named Body Parts will pop up. In this window, therapeutic protocols are organized with respect to areas in the body. At first, select your favorite area (anterior or posterior side of the body) so that the programs in that regard are listed. Then touch the desired program name so the Therapy window relevant to it opens. Push the Start button to start the program. Note that in this window you may alter the value of parameters to values of your choice other than those suggested by us. However, your changes will never be saved in the therapeutic protocols.


Hint: You may use the Search facility to find your desired therapeutic protocols. To do so, type the name or the number of programs in the Search section.

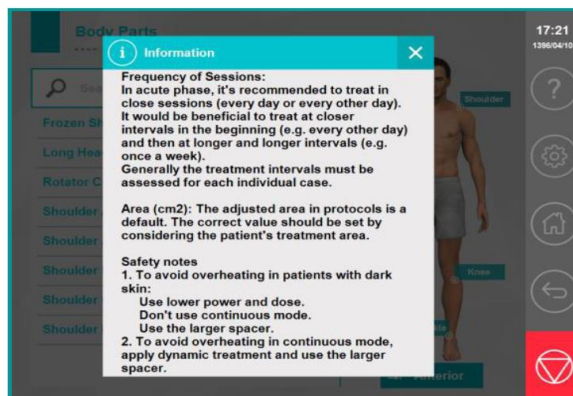Selecting the Information button  opens a window containing supplementary information such as therapy suggestions and safety advice.

If the Placement button  is touched, over the therapy menu , in the Hide menu, a window containing the placement images will be shown. Touching each of the thumbnail images on the right side of the window will display them in a larger scale on the left.

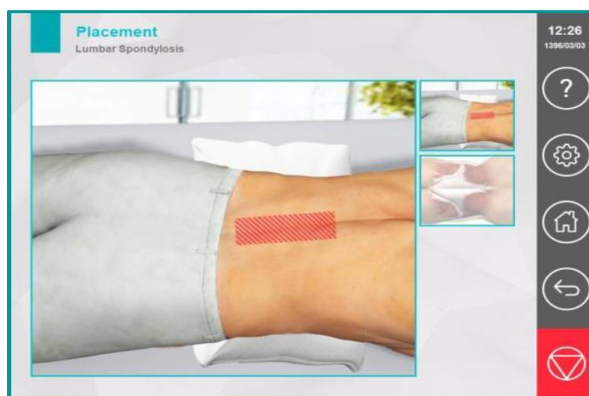Clinical Protocols

On the Home window, if Clinical Protocols is chosen, a window named Clinical Protocols will open. In this window, all the therapeutic protocols are organized in alphabetic order. Touch your desired program name so that the respective therapeutic protocol is shown. In this window, run the program by touching the Start button.

Note: when exerting therapy, if the Interlock connection with the device is disrupted one way or another, the output will stop and the error message “Interlock is not connected” will appear.


Warning: In therapies with the high power function, overheating the laser diode is expected. In case of observing the “Over Temperature” message, stop the therapy and wait awhile (at least 5 minutes) until the diode is cooled down. Right after the temperature reaches to an admissible value, you may continue the therapy.

Warning: Both the patient and the therapist must use Laser Safety Goggles to avoid risks of eye injuries.


Save As

To save a program with arbitrary parameters, adjust desired parameters first, in the Therapy window and then tap on Save As button  on the Hide menu  section. After doing so, a window pops up, asking for your typing the name of the program. When you are done typing it, push Save to close the window, and have the program saved in the User Programs.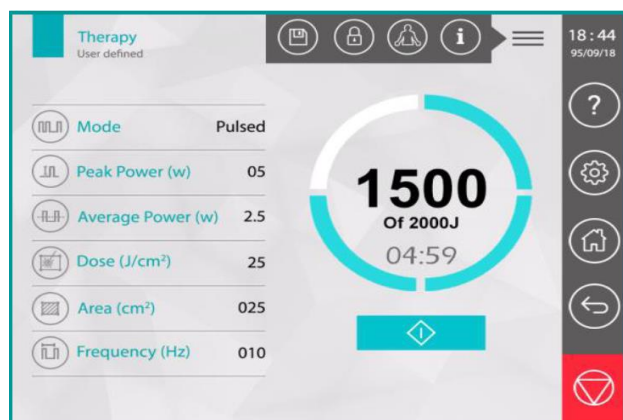


User Programs

Selecting User Programs in the Home page opens a window named User Programs. Programs saved by the user are shown in this window. Select the name of the desired program so that the respective Therapy window opens. Push the start to run the program.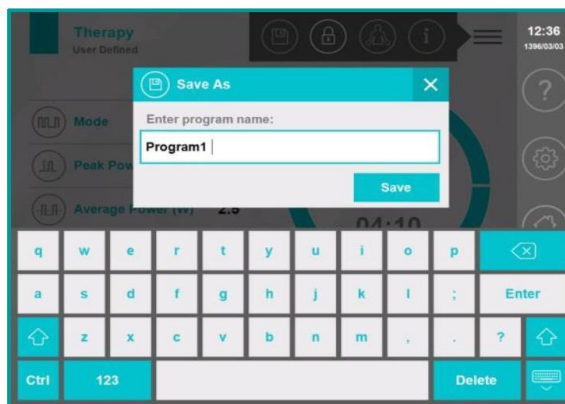


If you want to delete the program keep your finger on its name for a couple of seconds; in the opening window push Yes to clear it off the list.

As such, if you intend to correct or alter the therapy parameter values in a saved program, select the Edit button (✎) in front of the program name; accordingly Therapy Edit window turns up. Change the value of parameters in this window first, and then push Save to have the old values substituted with the new ones.

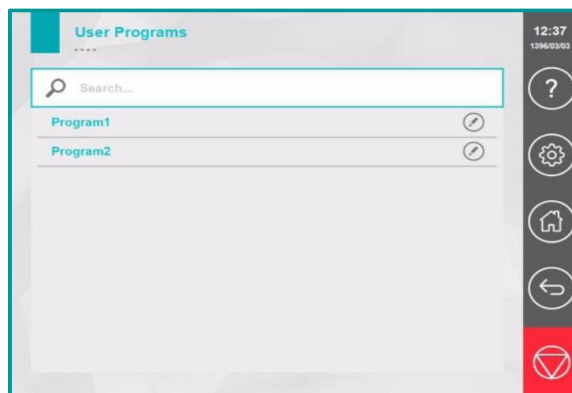
Selecting SETTINGS (⚙️), from the control bar on the right side of the screen, opens Settings. You may change the device adjustments including sound volume level, theme, font size, calibration test and ..., in this window.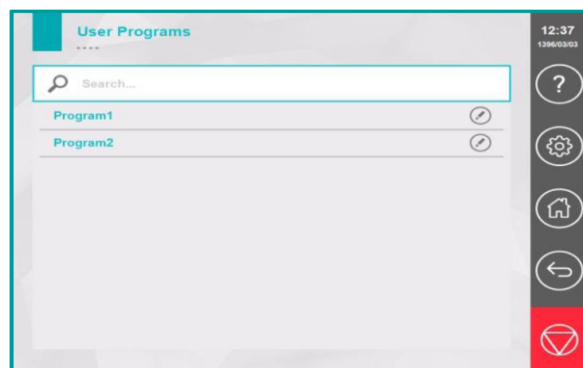


Settings/Calibration test

Selecting Handpiece, in the Settings window, opens a window named Settings/Handpiece.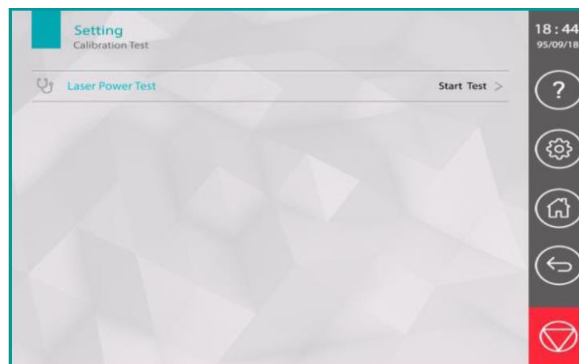

Selecting Calibration Test opens a window, in which the calibration procedure is shown. To do the test, insert the laser applicator into the calibration socket first; after that, push the Footswitch for 10 sec to initiate the laser emission. After calibration time is over (30 seconds), the result of the calibration test will be displayed in the bottom frame of the window. The scripts "Result: Limit" or "Result: Not OK" demonstrate that the output power is not calibrated and the device needs to be referred back to the After Sale's Service of the company in order to have its calibration process taken care of.

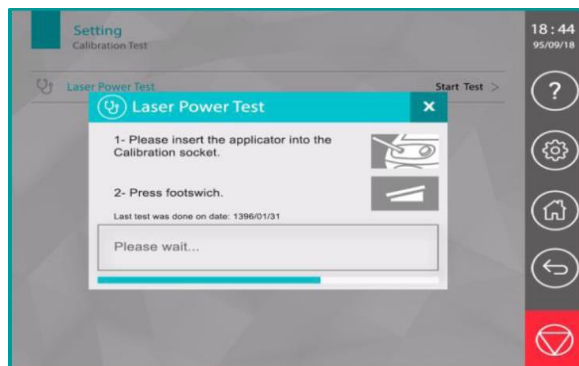Hint: make sure the protection glass on the tip of the handpiece and the glass at the bottom of the calibration socket are perfectly clean. Otherwise, wipe them clean first and then execute the test.

Hint: Do not release the Footswitch during the test since it causes the test to not finish properly.



Hint: Execute the calibration test every month to make sure of the authenticity and accuracy of the output.

Settings/Sound

Selecting Sound, in the Settings window, opens a window named Settings/Sound. Following parameters may be adjusted in this window:

Device Operation


- **Touch Sound Activated:** Touch Sound is a short beep that plays when touching a button. This can be deactivated via Touch Sound Activated; selecting this option, Touch Sound may be switched ON or OFF. This option is ON by default.
- **Alarm Volume:** Selecting Alarm Volume will open a window, in which the sound volume of the laser alert may be adjusted. Drag the slider bar to the right or left to adjust the sound volume.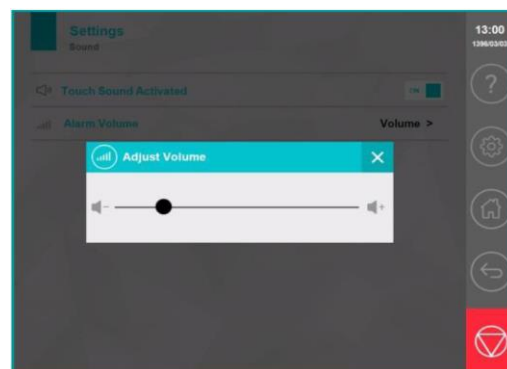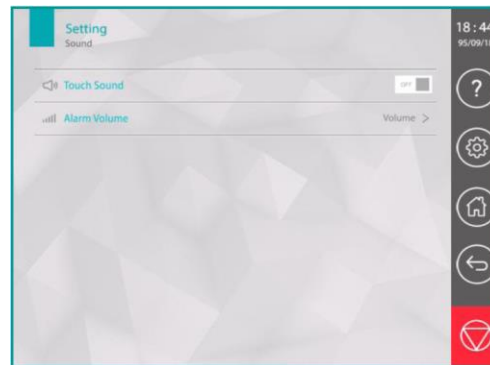

Settings/Reset

Selecting Reset, in the Settings window, will open a window named Settings/Reset. In this window, some options with the aim of resetting the device to its initial settings are provided, which are as follows:

- **Default Parameters:** Selecting this option resets therapeutic parameters in "Manual Operation/Therapy" to its factory settings.
- **Erase All Favorites:** Selecting this option deletes all programs defined as a favorite in the Favorite page.
- **Erase All User Programs:** Selecting this option deletes all programs saved in User Programs.
- **Default Settings:** This option is for resetting every device settings to its factory settings. These settings are as follows:
Theme, Font size, Therapist Nam, Password, Touch Sound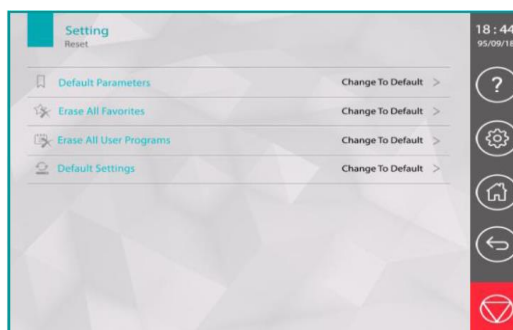

Settings/Display:

Selecting Display, in the Settings window, opens a window named Settings/Display. In this window, there are options for altering display adjustments as the user desires, which are as follows:

Theme: here 8 different themes exist.

Font Size: selecting this option opens a window, in which three display modes exist: Large, Medium and Small.

Therapist Name: Selecting this option opens a window along with a keyboard, where you are supposed to enter your desired name and save it. This name will be displayed on the top left of the Home page.


Date and Time: Time (24-hour format) and solar date (year/month/day) may be adjusted here. Time and date will be displayed on the top right of the control bar.

Settings/Security


Selecting Security, in the Settings window, opens a window named Settings/Security.

Selecting Password Set opens a window for changing the password. In this window, it is required to enter both old and new passwords. By default, the password is "0000."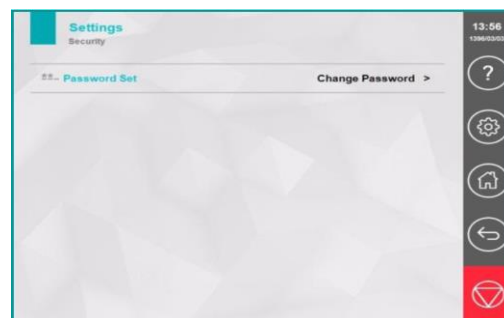

Settings/Update

Selecting Update, in the Settings window, opens a window named Settings/Update. Some options are provided in this window with the aim of updating the device software. To update the software, you may copy the new version of it on flash memory and then connect it to the USB port in the backside of the device; eventually, select Software Improvement to update it. If any error occurred during the updating process, select Software Restore to have the software reset to its initial version.

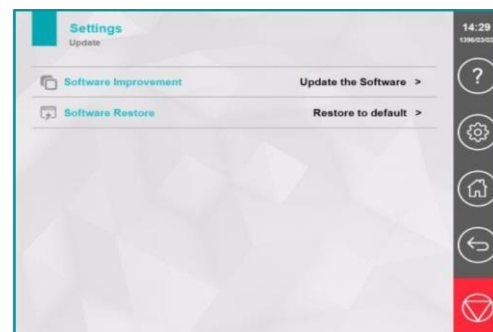Note: Updating the software may take 30 minutes to complete; do not unplug the power over this time.


Help

Selecting the Help button (?), from the control bar on the right, opens the Help window.

Help/Introduction


Select this option if you need help with how to work with the device. In Help/Introduction window using the table of content menu (☰), take a look at available headlines and choose your desired one to view the scripts relevant to it. Notice that contents may exceed one page; if so, swipe your finger to the left or right to move between the pages. You may move between the headlines via back (◀) and next (▶) buttons.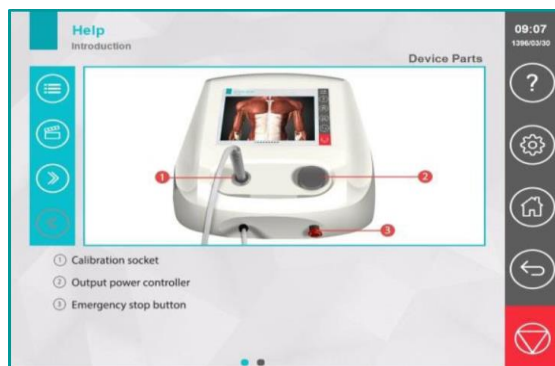

Selecting the movie (🎬) button plays a tutorial movie respectively.


Help/Laser Review

Selecting Laser Review, you may study contents about laser therapy modalities; these contents include modality introduction, physical basis, physiologic effects, applications, cases of precautions and etc.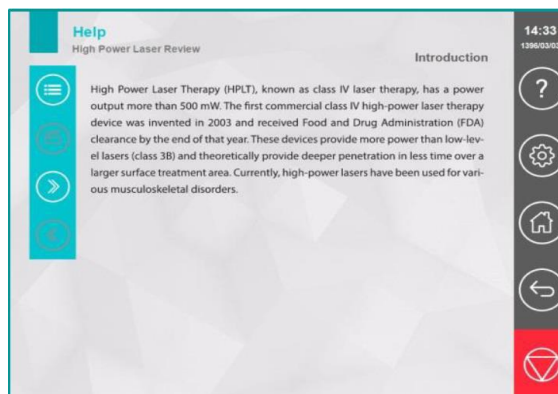

Help/Anatomy Library

Selecting Anatomy Library, you may see a set of useful anatomy images categorized based on body parts and in three levels: muscle, neuron and bone.

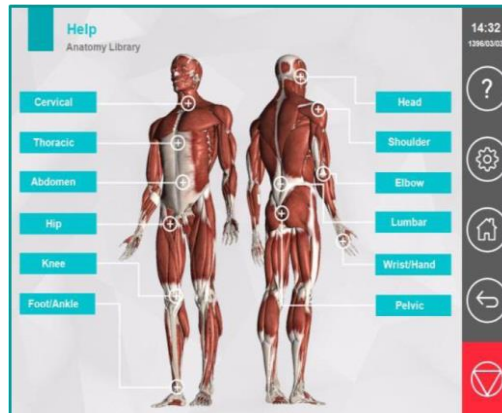THERAPEUTIC PROTOCOLS

- 1- Achilles Tendinitis
- 2- Acupuncture Points
- 3- Ankle Arthrosis
- 4- Brachialgia
- 5- Carpal Tunnel Syndrome
- 6- Cervical Muscle Spasm
- 7- Cervical Myofascial Trigger Points
- 8- Cervical Radiculopathy
- 9- Cervical Spondylosis
- 10- Coxalgia
- 11- Dupuytren's Contracture
- 12- Elbow Arthritis
- 13- Elbow Arthrosis
- 14- Elbow Bursitis
- 15- Elbow Myofascial Trigger Points
- 16- Epicondylitis 17- Fibromyalgia
- 18- Frozen Shoulder
- 19- Herpes Zoster
- 20- Hip Arthritis
- 21- Hip Arthrosis
- 22- Hip Bursitis
- 23- Hip Myofascial Trigger Points
- 24- Hip Tendinitis
- 25- Knee Arthrosis
- 26- Knee Bursitis
- 27- Knee Myofascial Trigger Points
- 28- Knee Tendinitis
- 29- Ligamentous Injury
- 30- Long Head of Biceps Tendonitis
- 31- Lumbar Muscle Spasm
- 32- Lumbar Myofascial Trigger Points
- 33- Lumbar Radiculopathy
- 34- Lumbar Spondylosis
- 35- Meniscus Injury
- 36- Metatarsalgia
- 37- Patellofemoral Pain Syndrome
- 38- Piriformis Syndrome
- 39- Plantar Fasciitis
- 40- Rotator Cuff Tendinitis
- 41- Sacroiliac Joint Pain
- 42- Sciatica
- 43- Scar
- 44- Shoulder Arthritis
- 45- Shoulder Arthrosis
- 46- Shoulder Bursitis
- 47- Shoulder Calcific Tendinitis
- 48- Shoulder Myofascial Trigger Points
- 49- Sprain
- 50- Temporomandibular Joint Pain
- 51- Tenosynovitis
- 52- Tibial Stress Syndrome
- 53- Trigeminal Neuralgia
- 54- Vascular Insufficiency
- 55- Wound
- 56- Wrist Arthritis
- 57- Wrist Arthrosis

WARRANTY AND RESPONSIBILITIES

Novin Med Medical Engineering provides a 24 months guarantee for the device from the date of purchase mentioned in the certificate of warranty. Additionally, it undertakes to promptly repair or replace the defective products referred to the company.

In order to use the service, the product should be referred to the after-sales service center of the company or its legal responsibility.

After-Sales Service Department:

No. 207, Persian Complex, Chahar Bagh-E-Bala Street, Esfahan, IRAN

Fax: 36267363

Tel: +98 31 36267172

Legal Responsible:

Name of Company: OBELIS S.A

Name of responsible person: Gideon Elkayam

Address: Boulevard General Wahis 53, 1030 Brussels, BELGIUM

Tel: +32.2.732.59.54

Fax: +32.2.732.60.03

E-Mail: mail@obelis.net

During device servicing, some of the information stored in the device memory may be lost; note that Novin Company does not take any responsibility for the stored data.

The warranty is valid only for the device itself and it does not include any other accessories of the package.

The warranty does not include:

- Device was repaired by unauthorized persons
- Improper use of the device, such as unauthorized power connection, static electricity damage, and so on.
- Any kind of failure due to improper maintenance or improper use.

Safety Notes

1. Patient and therapist in addition to all people in the room must wear Laser Safety Goggles to avoid eye injury.
2. In case of hazard, use the emergency stop button to quickly stop the output.
3. The device preferably should be used by a physiotherapist. If necessary, detach the Interlock from the device to prevent incompetent people from operating the device; doing so disables the activation of output laser.
4. Exert calibration test every month at "Settings /Calibration Test/Laser Power Test" to make sure about the authenticity and accuracy of the output.
5. In devices of class 4, the chance of eye injury is higher due to high power function; hence, necessary precautions must be taken in order to avoid possible hazards.
6. Avoid directing the laser beam to flammable substances (such as patient clothes, room curtain, sofa...) or flammable gases like alcohol.
7. It is advised to put warning signs of class 4 on the entering door of the room, where the laser device is located.

8. Avoid using the device in the vicinity of disturbing devices (such as mobile cell phone, diathermy, microwave, high power motors and so on) and adhere to the least distance of 5m. Device electricity phase of the power supply must be separate from that of disturbing devices and it must be checked out that their cables are not close.
9. To avoid the risk of the electric shock, the device must be connected to a socket that has earth grounding. Additionally, only replace a fuse with one of the exact same parameters.
10. When the power plug is connected, avoid opening the fuse box.
11. Avoid opening the device door.
12. Avoid pouring liquid substances on the body of the device.
13. The spacer has to be cleaned by ethyl alcohol 70% after each therapy session and before the beginning of the next therapy session so as to prevent the infection from spreading.

THERAPEUTIC ADVICE

- In case of patients with dark-colored skin do according to the following procedures to avoid overheating the skin and thermal damage:
 - Use lesser dose and power for such patients.
 - To the extent possible, do not use continuous mode.
 - Abate the area to which extent the laser is emitted, and respectively the power density on the skin, by connecting a big Spacer (of 35mm diameter) to the tip of the applicator.
- In chronic cases, increase the dose of therapy compared to the acute phase, from $5\text{J}/\text{cm}^2$ to $10\text{J}/\text{cm}^2$.
- When using the continuous mode, use the dynamic method to apply the output so as to prevent the skin from heating and thermal damage. To do so, first, connect a big Spacer (of 35mm diameter) to the tip of the applicator and then throughout the therapy session, steadily and gently move the applicator over the therapy area.
- In therapeutic protocols, the suggested value for the Area of therapy is just something default and suggested. The authentic value must be adjusted by the therapist with respect to the size of the patient's organ.

REPAIR AND MAINTENANCE

MAINTENANCE

- In order to prevent the device from warming up, make sure that there is no obstruction in front of the ventilation window.
- For a long-distance carriage of the device, disconnect the cables and insert them in the package, correctly.
- To clean the device and the applicator surface, use cotton and ethyl alcohol 70% (white alcohol) and never use strong solvents such as gasoline and thinner.
- Do not press the touch screen with objects other than your fingers especially sharp objects, such as pens, pencils, or nails.


TOUCH SCREEN DISPLAY (LCD)

The screen of the device is liquid crystal; therefore, the following precautions should be followed:

- Avoid any strong mechanical shock on LCD.
- Avoid placing the device in places with high temperatures such as the vicinity of radiators, hot air ventilators or direct sunlight.
- Avoid placing the device in areas where there is high mechanical vibration.
- Do not place the touch screen display in dusty or humid sites.
- Avoid placing the device exposed to a strong magnetic field.


APPLICATOR

Inside the applicator cable, there is an optical fiber that is sensitive to mechanical shock and bending, and in case of damage, it will obstruct the laser emission at the output. Thus, follow these instructions: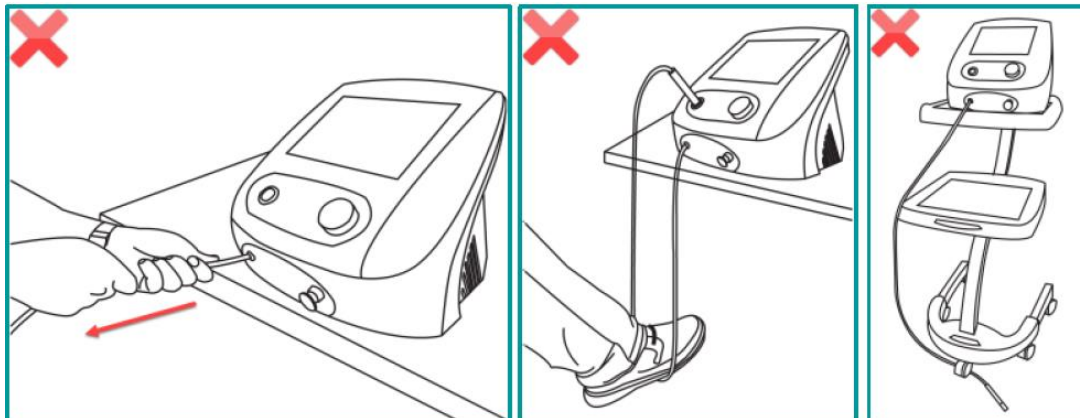

EMERGENCY STOP BUTTON


Never use this button, in normal conditions, for stopping the device. This key is meant for emergency occasions and frequent use may harm your device.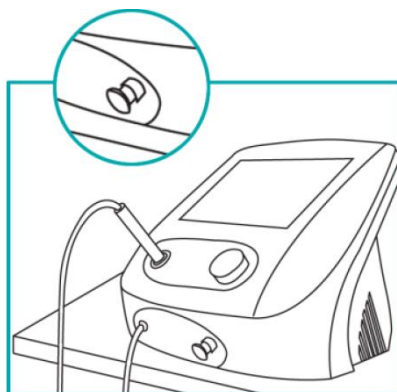

LASER PROTECTOR WINDOW


First and foremost, you have to prevent the applicator glass from becoming dirty because dirt reduces its lifetime.

Hold the applicator glass at an appropriate position and angle to the light so that dirt and particles on the glass are clearly visible. Use a cotton swab moistened with the cleansing solution accommodated with the device or by ethyl alcohol to clean it. Next, dry it immediately via the jet of dry compressed air. Be careful not to spray the cleansing solution directly inside the calibration socket or applicator. After cleaning, you must check the laser protection window again.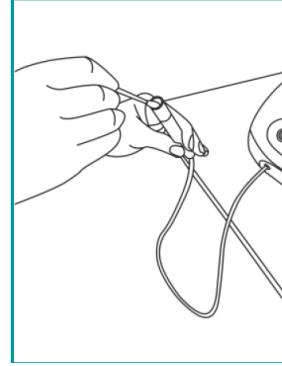


CALIBRATION SOCKET

Never activate laser emission by pushing the Footswitch or Start, unless when you are executing the calibration test which is a short time around 10 sec. Repetitive exerting this can do serious damage to the calibration test system.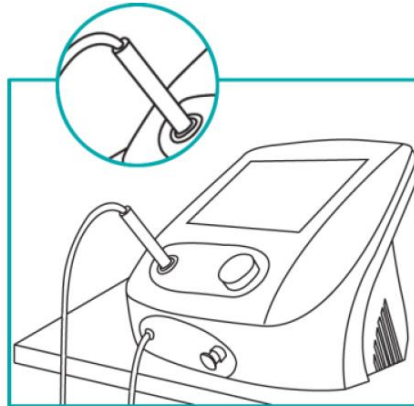

Use a cotton swab moistened with the cleansing solution accommodated with the device or by ethyl alcohol to clean up the glass at the bottom of the calibration socket and dry it right away by a dry clean cotton swab.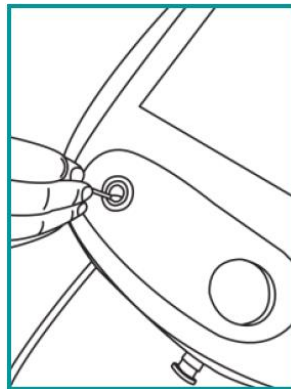


REPAIR AND TROUBLESHOOTING

If encountered any malfunctions while using the device, do as follows:

- 1- The device does not turn on:
 - a) Make sure the Emergency Stop Button is not pressed. If it is, un-press it and then switch the ON/OFF button to ON position.
 - b) Switch the power supply button to the ON position.
 - c) Make sure there is electricity in the outlets
 - d) Make sure the cable is intact
 - e) Make sure that the mains cable is properly inserted in the power outlet and the device connector is firmly plugged into the device port.
 - f) Turn off the device and pull the mains cable out of the power outlet and then check the fuses and make sure they are intact.

HOW TO OPEN THE FUSE BOX

- 2- Using the two edges on the left and right of the fuse box door, pull the door in a horizontal direction. Remove the fuses by using a screwdriver and replace any faulty fuses with 3A, 250V fuses. After that, push the box forward till it is fixed in its place.

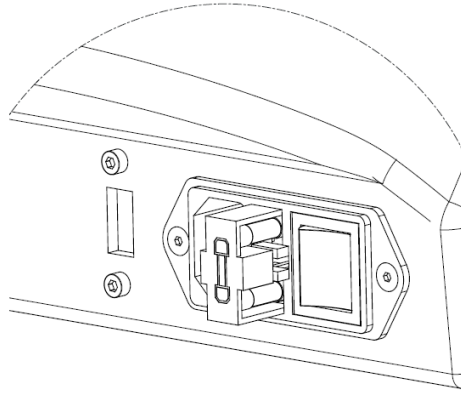- 3- There is a chance of the laser diode over-temperature in high power functions. If the message “Over Temperature” is shown, stop the therapy and wait awhile (at least 5 minutes) up until the temperature of the laser diode is reduced. When reached permissible temperature, continue your therapy again.

RECYCLING THE DEVICE

When the device expires, it has to be collected by legit companies that recycle electronic devices.

Note: The guide for this service isn't accommodated with the device by default, but if the customer requests, it will be given to them.

DEVICE SPECIFICATIONS

Output Parameters	
Laser Wavelength	810 + 980 nm
Laser Power	Up to 15W in Continuous mode
Guide Light Wavelength	650 nm
Guide Light Power	3 mW
Emission Modes	Continuous, Pulsed, Burst, Single
Laser beam emission area without a spacer	1 cm ²
Laser beam emission area with Ø25mm spacer	4.5 cm ²
Laser beam emission area with Ø35mm spacer	9 cm ²
Therapeutic Programs	
Number of Therapeutic Protocols	57 protocols
User Programs	1000 programs
Technical Data	
Input Power	Up to 10 VA
Mains Voltage	100-240V/50-60Hz
Mains Fuses	2 × Fast 3Amp/250V
Dimensions	360 mm × 270 mm × 270 mm (w × d × h)
Weight	5.5 Kg

STANDARDS AND CLASSIFICATION

Classification and Approvals	
Safety Class	Class I, Type BF, according to IEC 60601-1
Classification according to the MDD 93/42 EEC	Class IIb, according to Annex IX of MDD 2007/47/EC
Protection class of device against harmful ingress of water or particulate matter	IPX0, according to IEC 60601-1
Protection class of handpiece against harmful ingress of water or particulate matter	IPX0, according to IEC 60601-1
EMC related group and class	Group 1, Class B, according to Annex C of IEC 60601-1-2 & CISPR 11
Laser Class	Class 4
Standards	EN 60601-1:2005/A1:2012 EN 60601-1-2:2014 EN 60601-1-6:2010 EN 60601-2-22:2012 EN 60825-1:2014 EN 62304:2006 EN 62366:2008 EN ISO 14971:2012 EN 1041:2008 EN ISO 15223-1:2016 ISO 13485:2016 MEDDEV 2.7/1 rev4 MEDDEV 2.12-1 rev8

ENVIRONMENTAL CONDITIONS

Operating Temperature	-10 to +50°C
Operating Relative Humidity	5 to 85%
Transport/Storage Temperature	-10 to +50°C
Transport/Storage Relative Humidity	5 to 85%

Note: No modification of this device is allowed.

EMC DECLARATION

EMC Declaration for HPL 885G

Guidance and manufacturer's declaration – HPL 885G Emissions

The HPL 885G is intended for use in the electromagnetic environment specified below. The user of the HPL 885G should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Complies	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity

The HPL 885G is intended for use in the electromagnetic environment specified below. The user of the HPL 885G should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV Contact	± 8 kV Contact	
	± 2kV, ± 4kV, ± 8kV, ± 15kV Air	± 15kV Air	
Electrical fast transient/burst IEC 61000-4-4	Main Lines: ±2kV Signal Lines: ±1kV	Main Lines: ±2kV Signal Lines: ±1kV	
Surge IEC 61000-4-5	Ac power line: ±0.5,±1 kV line to line, ±0.5,±1, ,±2 kV line to ground dc power line: ±0.5,±1 kV line to line		
Voltage dips, short interruptions and voltage variations on power supply	95% for 0.5 period 95% for 1 period 30% for 25 periods 95% for 5 seconds	95% for 0.5 period 95% for 1 period 30% for 25	

EMC Declaration

input lines IEC 61000-4-11		periods 95% for 5 seconds
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m

NOTE UT is the a.c. mains voltage prior to application of test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The HPL 885G is intended for use in the electromagnetic environment specified below. The user of the HPL 885G should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Volts / 6 Volts for 150kHz to 80 MHz	3 Volts / 6 Volts	
Radiated RF IEC 61000-4-3	3 V/m for 80 MHz to 2.7 GHz	3 V/m for 80 MHz to 2.7 GHz	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM, and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the HPL 885G is used exceeds the applicable RF compliance level above, the HPL 885G should be observed to verify normal operation. If abnormal performance is observed, additional measures may necessary, such as re-orienting or relocating the HPL 885G.

b) Over the frequency range 150kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communication equipment and the HPL 885G

EMC Declaration

The HPL 885G is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The user of the HPL 885G can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the HPL 885G as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = 1.17\sqrt{P}$	$d = 1.17\sqrt{P}$	$d = 2.33\sqrt{P}$

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply to all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

SYMBOLS AND LABELS

SYMBOLS

-  IEC 60417-5333 Type B Applied Part
-  General warning, chance of hazard (physiological effect), refer to the function instructions
-  Referring to documents accommodated is necessary before installing
-  The device is not recyclable
-  USB Socket
-  Device Serial Number
-  Compliance with CE requirements
-  Manufacturer
-  Date of Manufacturing
-  European Authorized Representative
-  The button to select the Help window
-  The button to select settings window
-  The button to select the Home window
-  The button to go back to the previous page
-  Emergency Stop Button
-  Therapy with arbitrary parameters (Manual Operation)
-  Clinical Protocols
-  The list of therapeutic protocols with regard to body parts
-  User programs
-  Therapy Mode
-  Peak Power adjustment button
-  Average Power adjustment button

Symbols and Labels

	Dose adjustment button
	Area adjustment button
	Frequency adjustment button
	Hide Menu button
	Save favorite program
	Patient Lock
	Display placement images
	Display supplementary information for the therapy
	Start the therapy
	Search
	Calibration Test
	Sound adjustments
	Adjust sound volume of laser emission alarm (Alarm Volume)
	Reset the device setting to its initial state
	Service selection
	Select display settings
	Select security settings
	Update the software
	Introduction to device function
	Laser therapy guide review
	Anatomy Library
	About us
	Recently used
	Add a favorite program
	Edit the programs saved by the user
	Contents of Help menu

Symbols and Labels

Play the movie

Going to the next page in the Help menu

Going to the back page in the Help menu

LABELS

- Warning for Laser Radiation

- Warning for Laser Radiation of Safety Class 4

- Laser Output

- Emergency Stop


- Interlock

- Laser Aperture

- The Device Identification Label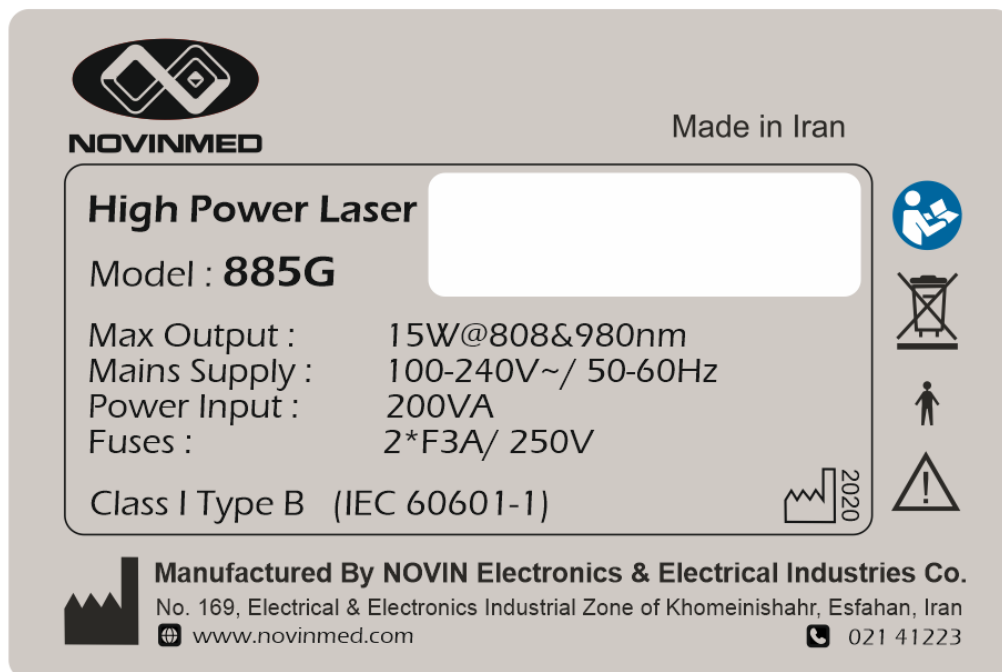

The label features the NOVINMED logo (two overlapping squares) and the text "NOVINMED" on the left, and "Made in Iran" on the right. The main text reads "High Power Laser" followed by a white rectangular box. Below this, the model is listed as "Model : 885G". Technical specifications include: "Max Output : 15W@808&980nm", "Mains Supply : 100-240V~/ 50-60Hz", "Power Input : 200VA", and "Fuses : 2*F3A/ 250V". The classification is "Class I Type B (IEC 60601-1)". On the right side, there are four icons: a person reading a manual, a crossed-out trash bin, a person silhouette, and a warning triangle. A small bar chart icon with the year "2020" is also present. At the bottom, it states "Manufactured By NOVIN Electronics & Electrical Industries Co." with the address "No. 169, Electrical & Electronics Industrial Zone of Khomeinishahr, Esfahan, Iran", the website "www.novinmed.com", and the phone number "021 41223".

DEVICE ACCESSORIES

No	Accessory name	Quantity
1	Mains Cable	1 unit
2	3A/250V fuse	2 units
3	Footswitch	1 unit
4	Interlock Connector	1 unit
5	Spacer of 25mm diameter	1 unit
6	Spacer of 35mm diameter	2 units
7	Laser Safety Goggles	2 unit
8	Quick Guide to 885G	1 unit
9	Guide to 885g (CD/DVD)	1 unit
10	Laser therapy book	1 unit
11	Final Test form	1 unit
12	Cleaning cloth for display	1 unit
13	Class 4 Laser Warning	1 unit
14	Tutorial movie for high power laser DVD	1 unit
15	Advisory Notes for 885G	1 unit
16	Jet of Dry Compressed Air	1 unit
17	Cleansing Solution Spray	1 unit
13	Class 4 Laser Warning	1 unit
14	Tutorial movie for high power laser DVD	1 unit
15	Advisory Notes for 885G	1 unit
16	Jet of Dry Compressed Air	1 unit
17	Cleansing Solution Spray	1 unit

TD400189.04
F71000RD10R02

NOVIN Electronics & Electrical Industries Co.

Manufactory

No. 169, Electrical and Electronics Industrial Zone of
Khomeynishahr, Esfahan, IRAN
ZIP Code: 8418148539

Office

No. 510, Parsian Complex, Chahar Bagh-E-Bala
Street, Esfahan (IRAN).
Tel: +98 31 36267172 Fax: +98 31 36267363

Customer Service

Tel: +98 31 31311544 Fax: +98 31 36267363

Web site: www.novinmed.com

E-Mail: info@novinmed.com